



ACTIVMOTION S PSI.

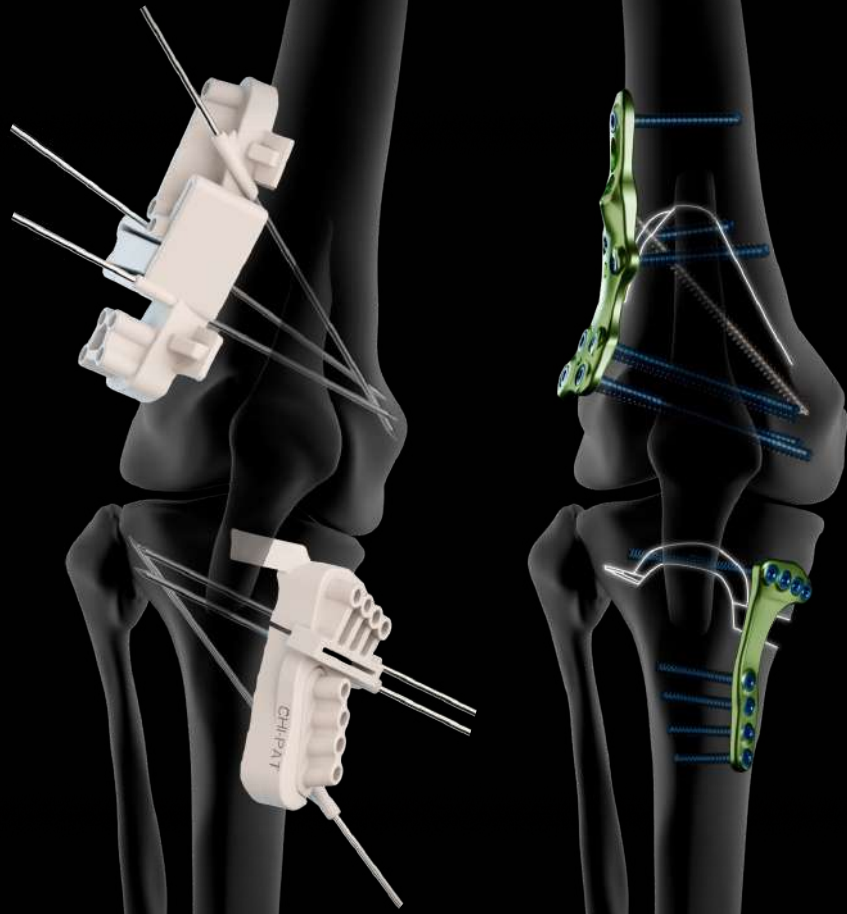

PATIENT-MATCHED
CUTTING GUIDES



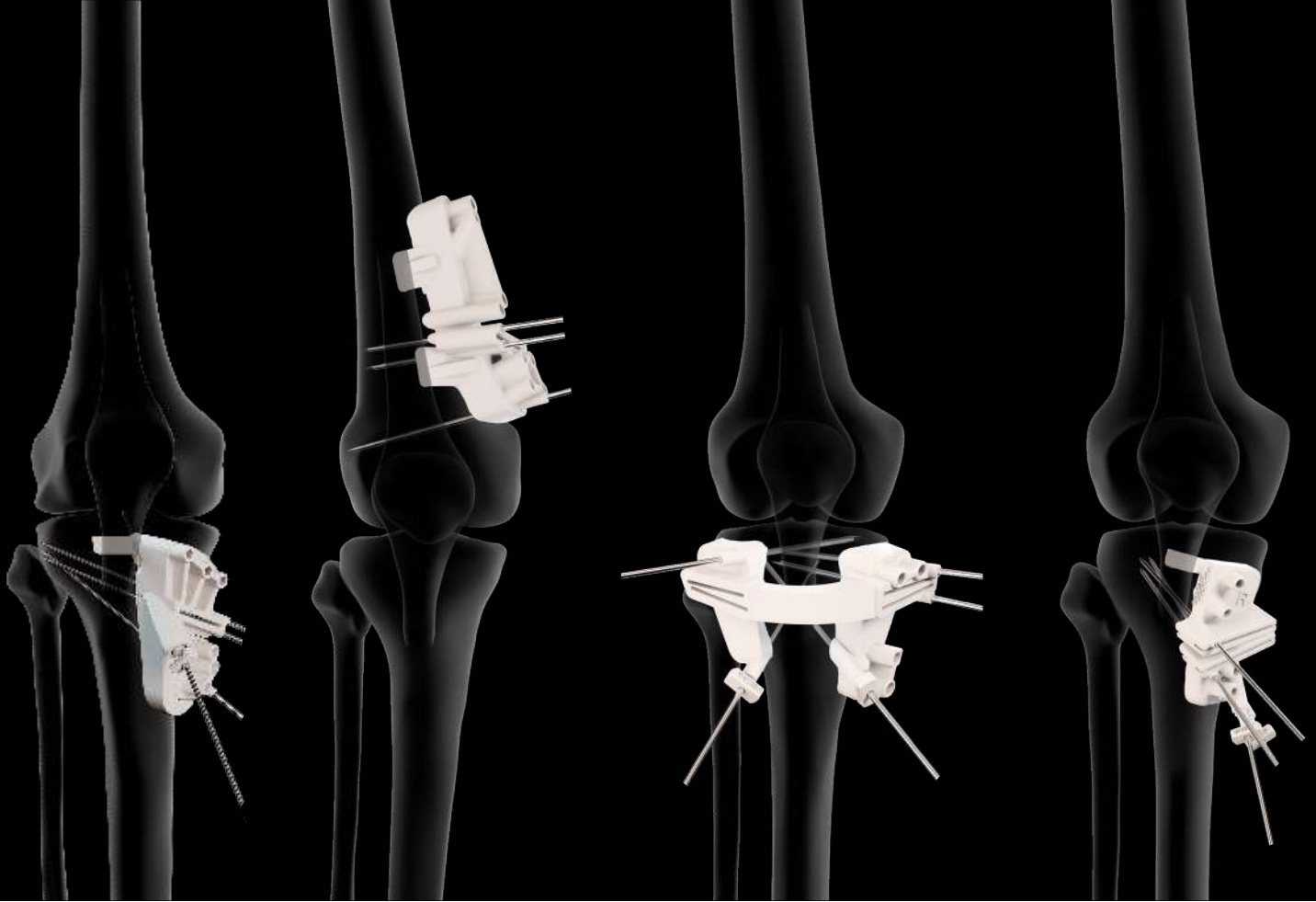

Activmotion S PSI.

PSI SOLUTION WITH ACTIVMOTION S

Intended purpose:


The implants of the Activmotion S range are intended for knee osteotomy in adults.
The patient-matched cutting guides are intended for guiding instruments.

Contraindications:


- Pregnancy.
- Acute or chronic local or systemic infections.
- Allergy to one of the materials used or sensitivity to foreign bodies.

Get the full picture.

FEMORAL CLOSING AND DEROTATION PLATES

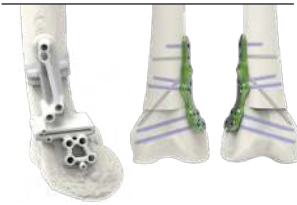


TIBIAL OPENING PLATES

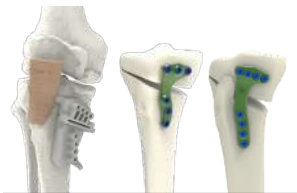


FEMORAL OPENING PLATES



TIBIAL CLOSING PLATES

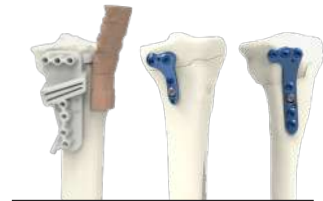

DEFLEXION PLATES

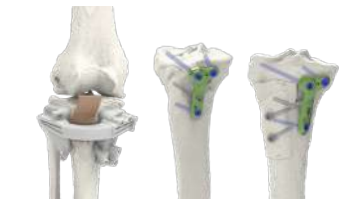



N.B.: Some of the patient-matched cutting guides and/or implants may not be available depending on your country.

Technical features.

PATIENT-MATCHED CUTTING GUIDE

The PSI solution is based on the CT scan of the patient and allows a correction of the knee in the three planes.

- The «**cutting K-wire(s)**»⁽¹⁾ allow(s):
 - to verify the direction of the cut(s);
 - to visualize the bone wedge to remove;
 - to confirm the position of the patient-matched cutting guide.
- The «**hinge K-wire(s)**»⁽²⁾ allow(s):
 - to stop the cut;
 - to protect the hinge during the opening;
 - to confirm the position of the patient-matched cutting guide;
 - to visualize the location and thickness of the hinge.
- **The anterior bracket**⁽³⁾ is positioned under the patellar tendon in line with its insertion.
- **The brackets**⁽⁴⁾ are congruent with the surface of the bone and allow positioning of the patient-matched cutting guide.
- **Drill guides**⁽⁵⁾: when the holes of the bone will align with the holes of the plate, the desired correction will be achieved in the three planes.
- Patient-matched cutting guide slot adapted to the surgeon's **sawblade**⁽⁶⁾.
- Patient-matched cutting guide slot for the **biplanar cut**⁽⁷⁾.

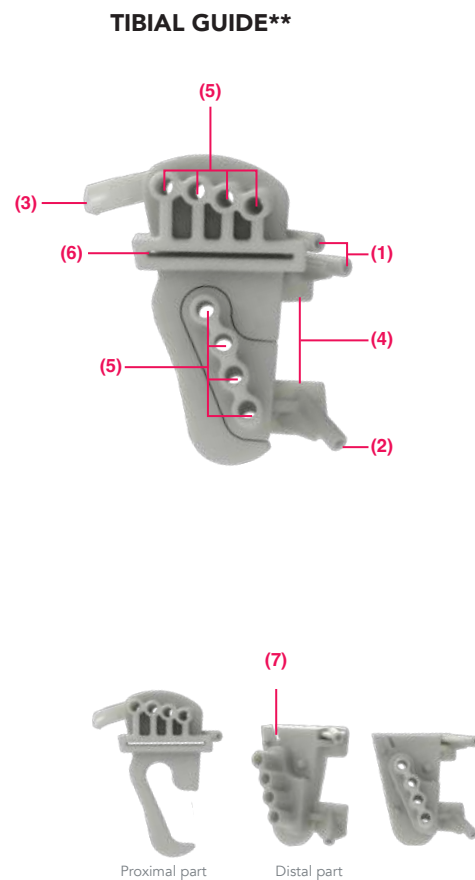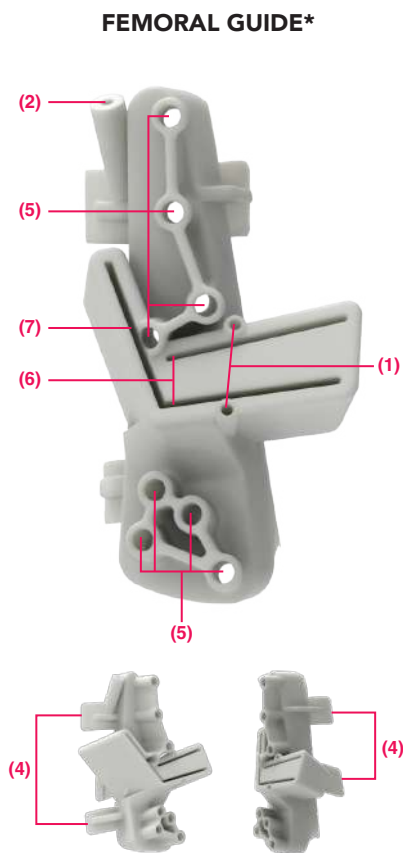

* Example of a Closing femoral guide associated to the JBTxM1D plate
 ** Example of a tibial guide associated to the ATxP2D plate

PATIENT-MATCHED CUTTING GUIDE - OPTIONS

The options shown are available on request and **must be selected** when creating a case on the dedicated portal.

TUNNEL FOR LIGAMENOTOPLASTY



ATT GUIDE



PATIENT-MATCHED WEDGE




FEMORAL HEAD CUTTING GUIDE*




ILIAC CREST CUTTING GUIDE*



BONE SUBSTITUTE CUTTING GUIDE*

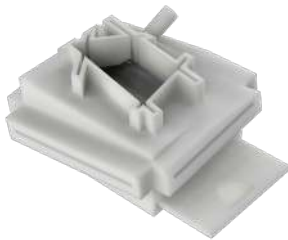

MEASURES

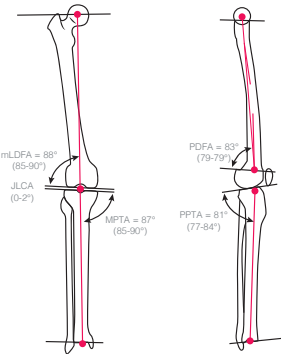

N.B.: Some of the patient-matched cutting guides options may not be available depending on your country.

PRECISION & REPEATABILITY OF THE CORRECTION

The ONE solution includes in its design the correction that will be applied and provides precision of the cut and plate positioning .

HOW IS THE PATIENT-MATCHED CUTTING GUIDE CREATED TO ENSURE THE PRECISION OF THE CORRECTION ON ALL THE PLANES?

Design steps:

Step 1: Requested correction applied on the preoperative bone



Step 2: plate applied at its precised position



Step 3: holes of the plate simulated on the postoperative bone



Step 4: plate removed



Step 5 - retro-engineering : bone positionned in the preoperative state





Step 6: patient matched cutting guide designed



SAFETY

The ONE solution brings safety to the key steps of your surgical procedure.

- Protection of the hinge by increasing fracture resistance.
- Pre-position of the plate and screws.

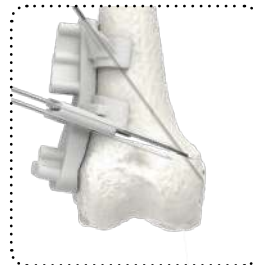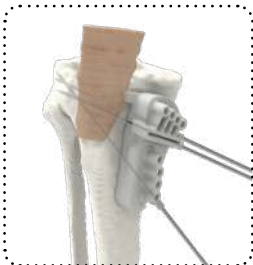


TIME SAVING IN THE OPERATING ROOM

The ONE solution allows to perform a precise correction. The solution includes in its design the correction that will be applied and the position of the plate at its optimized position.

The ONE solution reduces operating time when the learning curve has been reached, by using::



- Surgical technique already validated
- Position of the plate pre-established
- Screws lengths already calculated

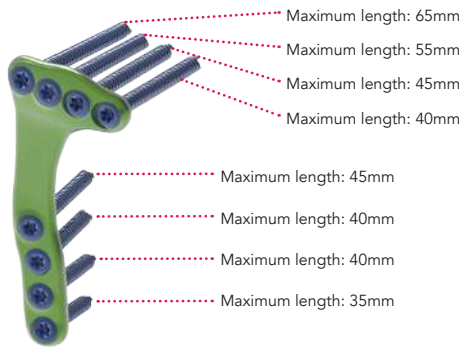

Example of information on the patient-matched surgical technique

REDUCTION OF THE NUMBER OF FLUOROSCOPY SHOTS

Fluoroscopy shots are only necessary to validate the position of the patient-matched cutting guide and to validate the final assembly*

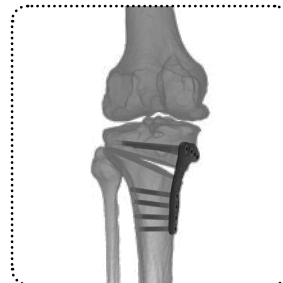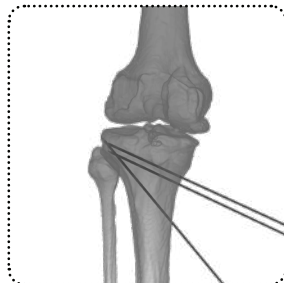

(*) when the learning curve has been reached.

ADAPTABLE TO DIFFERENT OSTEOTOMIES


The ONE solution allows to manage classic cases routinely, as well as to deal with complex surgeries.



Derotation



Slope change

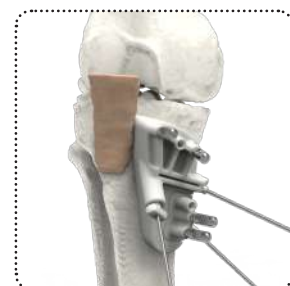

HTO with ligamentoplasty

Newclip PSI process.

HOW TO START THE PSI PROCESS ?


To start the process for a patient-matched cutting guide, you must give the information below:

- CT Scan (see Newclip protocol)
- Type of surgery
- Operative side
- Date of surgery
- Type of Plate
- Additional options

For more information, please consult our CT-Scan protocol:




CT SCAN PROTOCOL

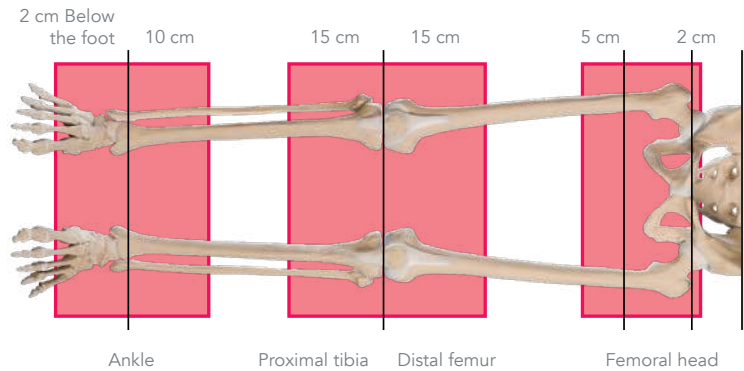

STEPS OF THE PSI PROCESS

FROM CT SCAN TO SURGERY IN MINIMUM 25 WORKING DAYS:

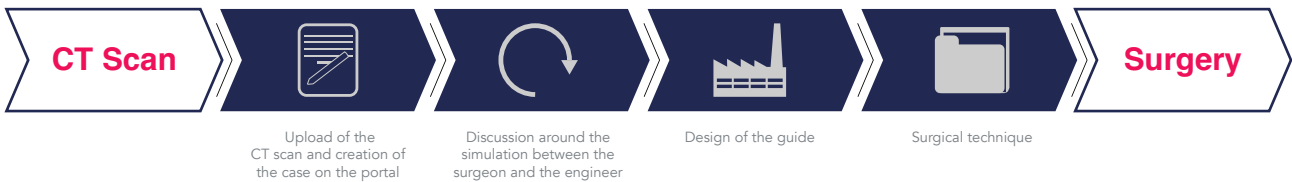

A DEDICATED PORTAL

WHY A WEB PORTAL ?

The ONE solution brings safety to the key steps of your surgical procedure.

- To request patient-matched cutting guides.
- To exchange documents (Medical imaging, patient-matched surgical technique, simulation file...).
- To follow the process.
- To communicate about the case.

In accordance with the personal data protection laws, the Provider will do its best efforts to take all useful precautions, in particular with regards to the nature of the Personal Data and the risks of the processing, to preserve the security and confidentiality of the Personal Data transmitted, processed or stored and to prevent their distortion, alteration, damage, accidental or unlawful destruction, loss disclosure and/or access by third parties not previously authorized.



ONE platform

Surgical technique.

MOWHTO WITH BIPLANAR CUT - GUIDE POSITIONING (STEP 1/3)

The following surgical technique illustrates the Activmotion S medial opening wedge high tibial osteotomy plate size 2 (ATxP2D). The same steps apply to the Activmotion S medial opening wedge high tibial osteotomy plate size 1 (ATxP1D).



Guide positioning:

Position the patient-matched (PM) cutting guide (ANC656) on the antero-medial surface of the tibia using the congruence of the PM cutting guide and the anterior and medial brackets.


Caution: Please do not modify the insertion of the patellar tendon.



«Cutting K-wire» insertion:


Insert a Ø2.2 mm K-wire (33.0222.200) in the «cutting K-wire» hole positioned on the medio-proximal bracket.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

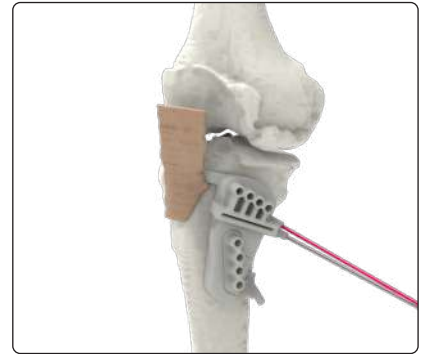

Optional - second «cutting K-wire»:

A second Ø2.2 mm K-wire (33.0222.200) can be inserted above the first one.

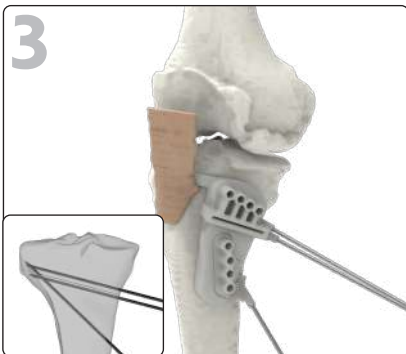

«Hinge K-wire» insertion:

Insert a Ø2.2 mm K-wire (33.0222.200) in the «hinge K-wire» hole positioned on the medio-distal bracket.


To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

Check the position of the PM cutting guide by comparing the fluoroscopy shot with the PM surgical technique.



Drill the holes:

Drill bicortically with the Ø4.0 mm drill bit (ANC211) through the 4 distal holes of the PM cutting guide (3 holes for the ATxP1D) and monocortically (ANC211) through the 4 proximal holes (3 holes for the ATxP1D).

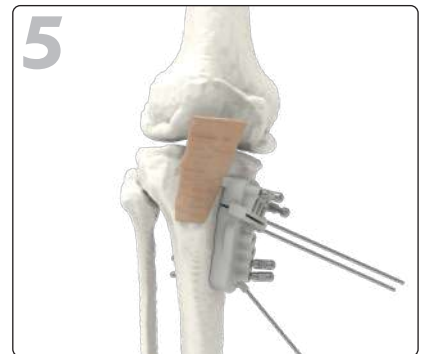

Fixation pins insertion:

Insert four long pins (ANC657), two in the proximal part of the PM cutting guide and two distal, by using a mallet.

MOWHTO WITH BIPLANAR CUT - CUTTING (STEP 2/3)



Initiate the cut:


Insert the sawblade in the PM cutting guide slot and initiate the cut in the antero-posterior direction.



Remove the proximal part of the PM cutting guide:

Remove the proximal long pins (ANC657) and the second «cutting K-wire» (33.0222.200).


Then remove the proximal part of the PM cutting guide (ANC656).

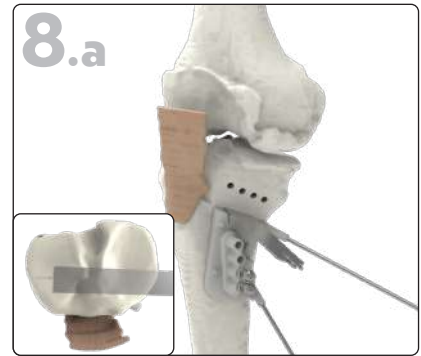

Finalize the cut:

Reinsert the sawblade and finalize the cut in medio-lateral direction.

Once the sawblade reaches the «hinge K-wire», the cut is complete.

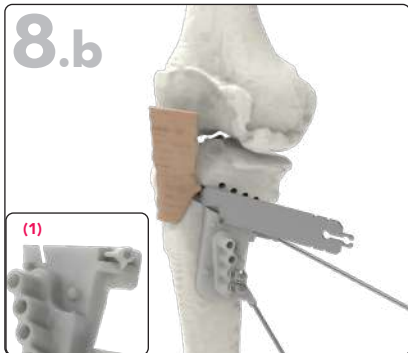

Biplanar cut:

Insert the sawblade in the biplanar slot⁽¹⁾. The cut must be bicortical.



Remove the distal part of the PM cutting guide:

Remove the distal long pin (ANC657) and the «cutting K-wire» (33.0222.200). Then remove the distal part of the PM cutting guide (ANC656) by sliding it over the «hinge K-wire».



Perform the opening:

Insert the patient-matched wedge (ANC956) or a metallic wedge for knee osteotomy (ANC019 to ANC023 and ANC025), to perform the opening.

N.B: The patient-matched wedge is an option and **must be selected** when the case is uploaded on the portal.

MOWHTO WITH BIPLANAR CUT - PLATE FIXATION (STEP 3/3)



11.a
Plate positioning:
 Insert two short pins (ANC774) into the two distal holes of the plate.



11.b
Plate positioning:
 The planned osteotomy is reached when the holes of the bone are facing the holes of the plate. Insert two short pins (ANC774) in the most lateral and the most medial proximal holes.



12
Plate fixation:
 Press the plate against the bone and use the countersink (ANC120-US) to prepare the first cortical of the two remaining holes.



ANC120-US

Insert two locking screws (ST4.5LxD) using the screwdriver (ANC975) without locking them.



ANC975



13
Finalize the assembly:
 Finalize the assembly by inserting the remaining Ø4.5 mm locking screws (ST4.5LxD).

A screw must be inserted right after removing a short pin (ANC774).

Proceed by inserting alternatively proximal and distal screws getting further and further away from the osteotomy site. Use the countersink (ANC120-US) to prepare the first cortical.

Then, remove the patient-matched wedge (ANC956) and the hinge K-wire (33.0222.200).

FINAL RESULT.




ACL TUNNEL OPTION*


* The ligamentoplasty tunnel is an option and must be selected when the case is uploaded on the portal.

The following surgical technique illustrates the Activmotion S medial opening wedge high tibial osteotomy with ligamentoplasty plate (ALTxP1D). The same steps apply to the Activmotion S opening wedge high tibial osteotomy with ligamentoplasty and suture button plate (AETxP1D).


N.B: The surgical technique for a MOWHTO with ligamentoplasty follows the same first steps as the MOWHTO surgical technique from 1 through to 6 (see pages 9 and 10).

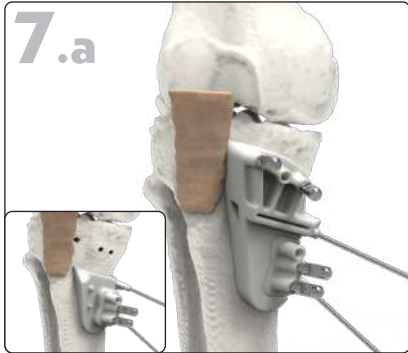

Remove the proximal part of the patient-matched cutting guide:
Remove the proximal long pins (ANC657). Then remove the proximal part of the patient-matched (PM) cutting guide (ANC656).

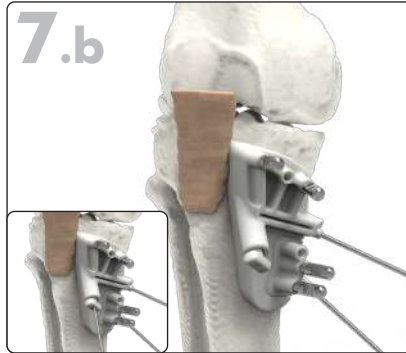

Position the second proximal part of the PM cutting guide:
Position the second proximal part of the PM cutting guide (ANC656) with the ACL tunnel and fix it with long pins (ANC657). Insert the holding plot (ANC1350) and a Ø2.2 mm K-wire (33.0222.200) into it.

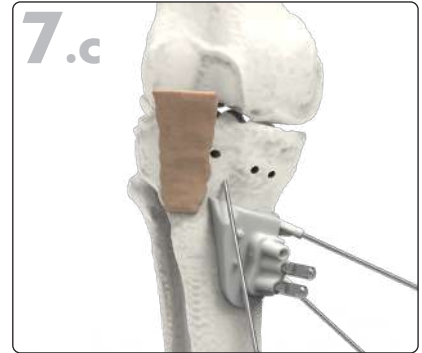

Remove the second proximal part of the PM cutting guide:
Remove the proximal long pins (ANC657), the holding plot (ANC1350) and then remove the second proximal part of the PM cutting guide (ANC656) by sliding it over the ligamentoplasty K-wire.

N.B: The next steps of the surgical technique for a MOWHTO with ligamentoplasty, follow the same steps as the MOWHTO surgical technique from 8 to 12 (see pages 8 and 9).



Finalize the assembly:
Finalize the assembly by inserting the remaining Ø4.5 mm locking screws (ST4.5LxxD).
A screw must be inserted right after removing a short pin (ANC774).
Proceed by inserting alternatively proximal and distal screws getting further and further away from the osteotomy site. Use the countersink (ANC120-US) to prepare the first cortical.
Remove the patient-matched wedge (ANC956).

FINAL RESULT.



Remove the hinge K-wire (33.0222.200).
With the help of the ligamentoplasty K-wire (33.0222.200) previously placed, perform the ACL tunnel with your standard instruments.

MCWHTO - GUIDE POSITIONING (STEP 1/3)


The following surgical technique illustrate the Activmotion S medial closing wedge high tibial osteotomy plate size 2 (BTxMD2D). The same steps apply to the Activmotion S medial closing wedge high tibial osteotomy size 1 (BTxMD1D), and to the Activmotion S lateral closing wedge high tibial osteotomy size 1 (BTxBD1D) and size 2 (BTxBD2D).



Guide positioning:

Position the patient-matched (PM) cutting guide (ANC656) on the antero-medial surface (lateral surface for the lateral closing wedge) of the tibia using the congruence of the PM cutting guide and the anterior and medial brackets.


Caution: Please do not modify the insertion of the patellar tendon.

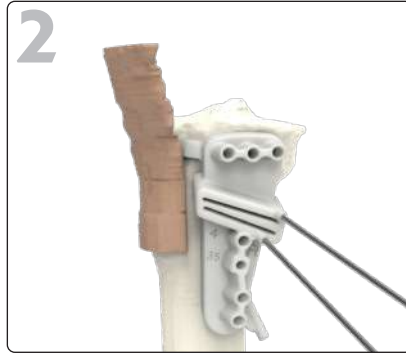

«Cutting K-wire» insertion:

Insert the Ø2.2 mm K-wires (33.0222.200) in the «cutting K-wire» holes.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

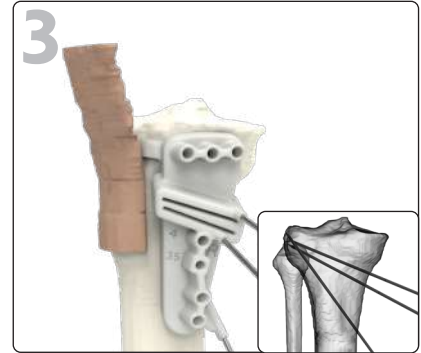

«Hinge K-wire» insertion:

Insert the Ø2.2 mm K-wire (33.0222.200) in the «hinge K-wire» hole.


To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

Check the position of the PM cutting guide by comparing the fluoroscopy shot with the PM surgical technique.




Drill the Ø4.0 holes:

Drill bicortically with the Ø4.0 mm drill bit (ANC211) through the 3 distal round shape holes (3 holes for the BTxBD2D, and 1 hole for the size 1 BTxMD1D and BTxBD1D), and monocortically (ANC211) through the 3 proximal holes (3 holes for the BTxBD2D, and 2 holes for the size 1 BTxMD1D and BTxBD1D).

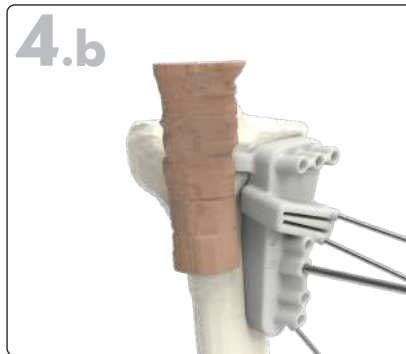

Drill the Ø3.5 holes:


Drill bicortically with the Ø3.5 mm drill bit (ANC1075) through the 1 distal square shape hole.

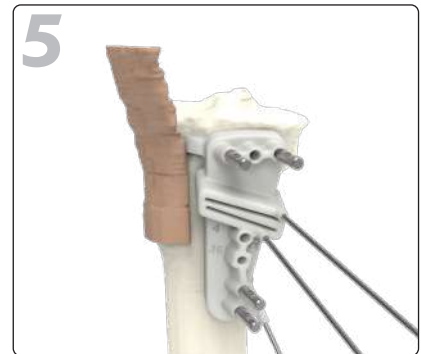


Fixation pins insertion:

Insert four long pins (ANC657), two in the proximal part of the PM cutting guide and two in the most distal holes (two for the BTxBD2D but only one in distal for the size 1 BTxMD1D and BTxBD1D), by using a mallet.

MCWHTO - CUTTING (STEP 2/3)

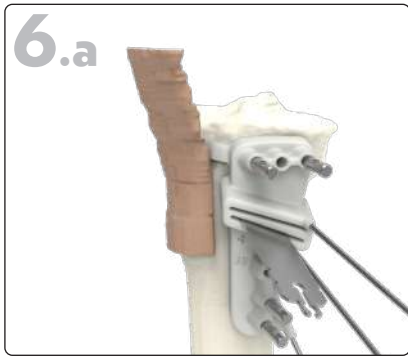

Initiate the cuts:
Insert the sawblade in the distal PM cutting guide slot and initiate the cut.

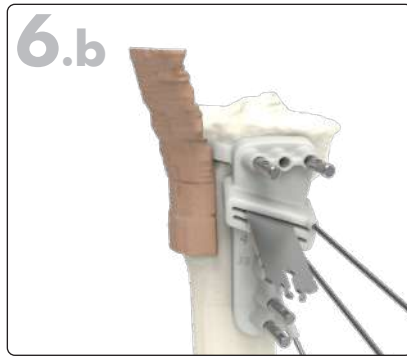

Initiate the cuts:
Insert the sawblade in the proximal PM cutting guide slot and initiate the cut.



Remove the PM cutting guide:
Remove the long pin (ANC657) and the «cutting K-wires» (33.0222.200). Then remove the PM cutting guide (ANC656) by sliding it over the «hinge K-wire».



Finalize the cuts:
Reinsert the sawblade in the previous cuts and finalize the cuts.

Once the sawblade reaches the «hinge K-wire», the cuts are complete.



Remove the bone wedge:
Remove the bone wedge and remove the «hinge K-wire» (33.0222.200).

MCWHTO - PLATE FIXATION (STEP 3/3)



Plate positioning:

Insert two short pins (ANC774) into the two distal holes of the plate (two for the BTxBD2D and one for the size 1 BTxMD1D and BTxBD1D).



Plate positioning:

The planned closing is reached when the holes in the bone are facing the holes of the plate. Insert two short pins (ANC774) in the most lateral and most medial proximal holes.



Plate fixation:

Press the plate against the bone and use the countersink (ANC120-US) to prepare the first cortical of the two remaining holes.



ANC120-US

Insert two locking screws (ST4.5LxD) using the screwdriver (ANC975) without locking them.



ANC975

Insert the Ø4.5 mm non-locking screw (CT4.5LxD-ST) using the screwdriver (ANC975) in the ramp oblong hole.

FINAL RESULT.



Finalize the assembly:

Insert the remaining Ø4.5 mm locking screws (ST4.5LxD).

A screw must be inserted right after removing a short pin (ANC774).

Proceed by inserted alternatively proximal and distal screws getting further and further away from the osteotomy site. Use the countersink (ANC120-US) to prepare the first cortical.

ACWHTO SUPRA-TUBERCLE - GUIDE POSITIONING (STEP 1/3)


The following surgical technique illustrate the Activmotion S anterior closing wedge high tibial osteotomy plate size 1 (BATxMD1D) with a supra-tubercle technique.



Guide positioning:

Position the patient-matched (PM) cutting guide (ANC656) on the anterior surface of the tibia using the congruence of the PM cutting guide.


Warning: Please do not modify the insertion of the patellar tendon.

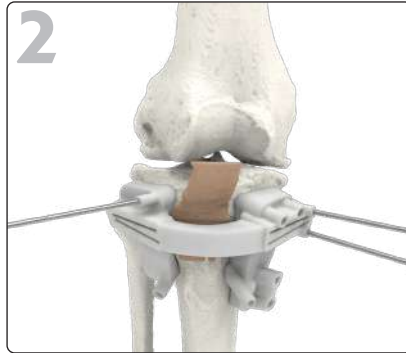

«Cutting K-wires» and «positioning K-wire» insertion:

Insert a Ø2.2 mm K-wire (33.0222.200) in the «cutting K-wire» holes and in the «positioning K-wire» hole.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

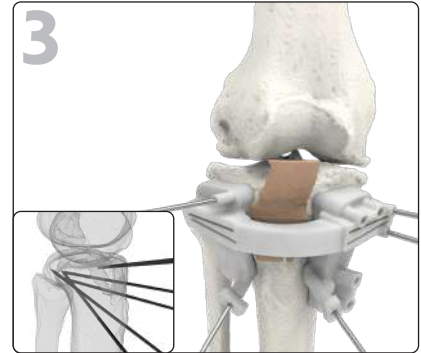


«Hinge K-wire» insertion:

Insert the holding plots (ANC1350) in the «hinge K-wire» holes and then a hinge K-wire (33.0222.200) into it.


To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

Check the position of the PM cutting guide by comparing the fluoroscopy shot with the PM surgical technique.

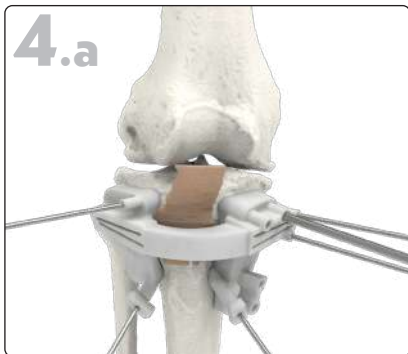

Drill the Ø 4.0mm holes:

Drill monocortically with the Ø4.0 mm drill bit (ANC211) through the proximal holes of the PM cutting guide and bicortically (ANC211) through the round shape distal hole.

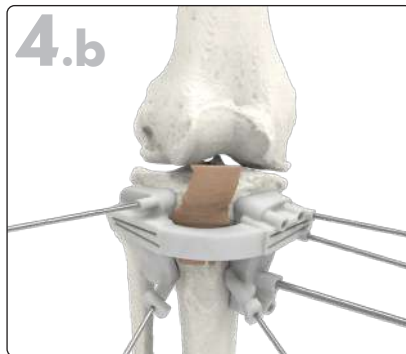

Drill the Ø 3.5mm holes:


Drill bicortically with the Ø3.5 mm quick coupling drill bit (ANC1075) through the square shape distal hole.



Fixation pins insertion:

Insert three long pins (ANC657), two in the proximal part of the PM cutting guide and one in the distal hole, by using a mallet.


ACWHTO SUPRA-TUBERCLE - CUTTING (STEP 2/3)

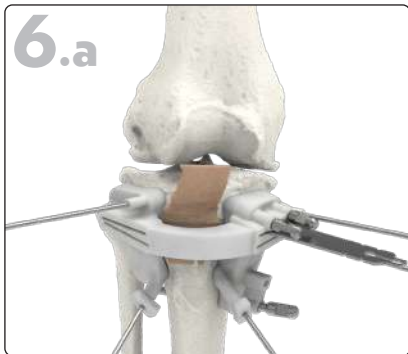

6.a

Initiate the cut:

Insert the sawblade in the PM cutting guide slots in the medial part of the PM cutting guide and initiate the cut.

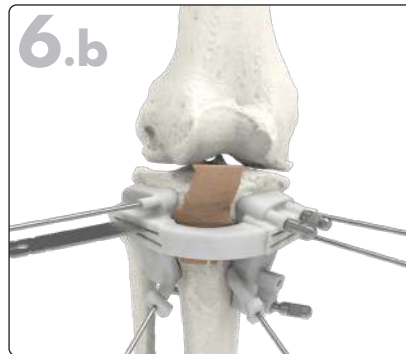

6.b

Initiate the cut:

Insert the sawblade in the PM cutting guide slots in the lateral part of the PM cutting guide and initiate the cut.




7

Remove the PM cutting guide:

Remove the long pins (ANC657), the «cutting K-wires» (33.0222.200), «the positioning K-wire» (33.0222.200) and the holding plots (ANC1350).

Then remove the PM cutting guide (ANC656) by sliding it over the «hinge K-wires».

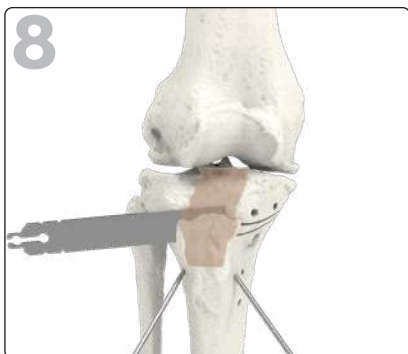

8

Finalize the cut:

Perform the biplanar cut by hand. Reinsert the sawblade and finalize the cuts. A slice of bone must be removed for the biplanar cut to allow the osteotomy closure.

Once the sawblade reaches the «hinge K-wires» the cuts are complete.




9


Remove the bone wedge:

Remove the bone wedge and remove the «hinge K-wires» (33.0222.200).

ACWHTO SUPRA-TUBERCLE - PLATE FIXATION (STEP 3/3)

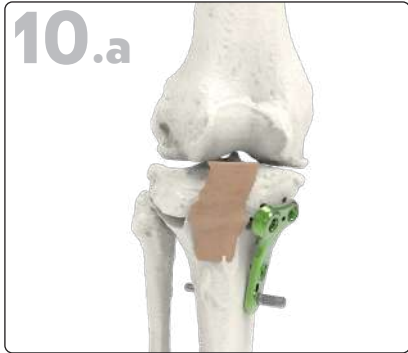

10.a
Plate positioning:
 Insert a short pin (ANC774) into the distal hole of the plate.

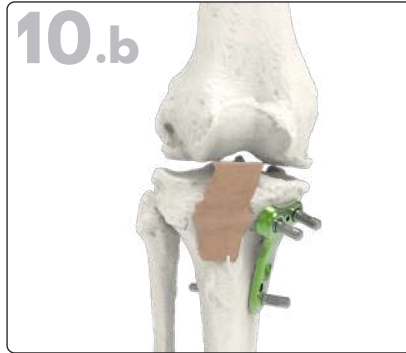

10.b
Plate positioning:
 The planned closing is reached when the holes of the bone are facing the holes of the plate. Insert two short pins (ANC774) in the proximal holes.




11
Plate fixation:
 Remove the most anterior short pin (ANC774). Press the plate against the bone and use the countersink (ANC120-US) to prepare the first cortical.



Insert a locking screw (ST4.5LxD) using the screwdriver (ANC975).



Perform the same steps for the second proximal short pin (ANC774).

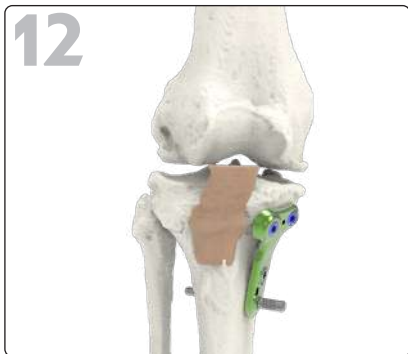

12
Finalize the assembly:
 Finalize the assembly by inserting a nonlocking screw in the compression hole (CT4.5LxD) thanks to the screwdriver (ANC975).

Replace the last remaining short pin (ANC774) by a locking screw (ST4.5LxD).

Please, use the countersink (ANC120-US) to prepare the first cortical.

FINAL RESULT.



ACWHTO TRANS-TUBERCLE - TRANSPOSITION OF THE ATT (STEP 1/4)


The following surgical technique illustrate the Activmotion S anterior closing wedge high tibial osteotomy plate size 1 (BATxMD1D) with a trans-tubercle technique.



Guide positioning:

Position the patient matched (PM) cutting guide (ANC656) for the elevation of the anterior tibial tubercle on the antero-medial surface of the tibia using the congruence of the PM cutting guide.

Warning: Please do not modify the insertion of the patellar tendon.

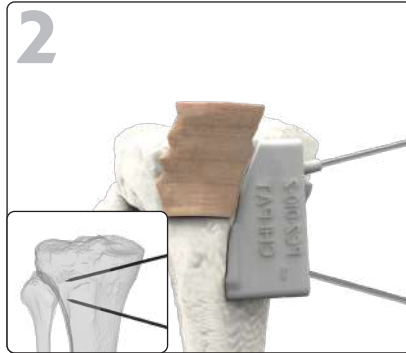


«Positioning pins» insertion:

Insert Ø2.2 mm K wires (33.0222.200) in the «positioning K-wires» holes.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

Check the position of the PM cutting guide by comparing the fluoroscopy shot with the PM surgical technique.

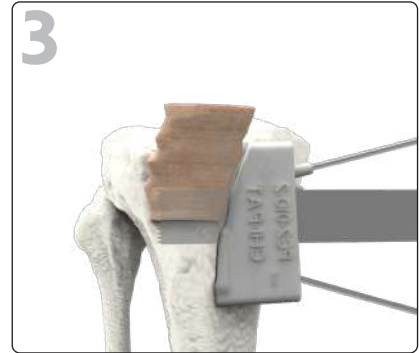

Initiate the cut:

Insert the sawblade in the PM cutting guide slot and initiate the cut.



Remove the guide:

Remove the «positioning K-wires» (33.0222.200) and the PM cutting guide (ANC656).



Finalize the cut:

Reinsert the sawblade and finalize the cut in proximal and distal, the cut must be bicortical.

Then realize the elevation of the ATT.

ACWHTO TRANS-TUBERCLE - GUIDE POSITIONING (STEP 2/4)



Guide positioning:


Position the PM cutting guide (ANC656) on the antero-medial surface of the tibia using the congruence of the PM cutting guide and the brackets.



«Cutting K-wires» insertion:

Insert the Ø2.2 mm K-wires (33.0222.200) in the «cutting K-wire» holes.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

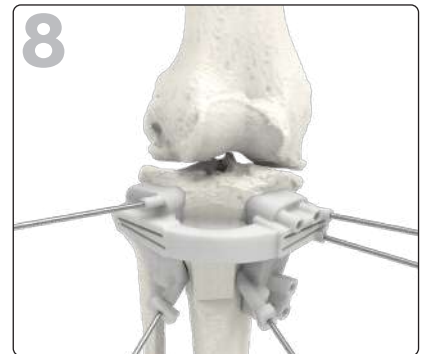

«Hinge K-wires» insertion:

Insert the holding plots (ANC1350) in the «hinge K-wire» holes and insert the Ø2.2 mm K-wires (33.0222.200).

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

Check the position of the PM cutting guide by comparing the fluoroscopy shot with the PM surgical technique.

N.B: Steps 9 to 10 for the ACWHTO trans-tubercle follow the steps 4 to 5 of the ACWHTO supra-tubercle (see page 16) .

ACWHTO TRANS-TUBERCLE - CUTTING AND PLATE FIXATION

(STEPS 3 & 4/4)



Initiate the cuts:

Insert the sawblade in the PM cutting guide slots to initiate the cuts.

N.B: Steps 12 to 17 of the ACWHTO trans-tubercle follow the steps 7 to 12 of the ACWHTO supra-tubercle (see pages 17 & 18).

FINAL RESULT.



Reattach the Anterior tibial tubercle and fix it with two non-locking screws (H1.4QT4.0Lxx-ST).

Please use the countersink (ANC120-US) to prepare the first cortical

ACWHTO INFRA-TUBERCLE

The following surgical technique illustrate the Activmotion S anterior closing wedge high tibial osteotomy plate size 2 (BATxMD2D) with an infra-tubercle technique



N.B.: Steps 1 to 9 of the ACWHTO infra-tubercle follow the steps 6 to 14 of the ACWHTO trans-tubercle (see page 20)



Guide positioning:

Insert a short pin (ANC774) into the proximal hole of the distal part of the plate. Remove the «hinge K-wires».

Warning: Please note that the most distal and proximal holes are not yet drilled in order to reduce the size of the PM cutting guide.



Plate positioning:

The planned closing is reached when the holes of the bone are facing the holes of the plate. Insert two short pins (ANC774).



Plate fixation:

Press the plate against the bone and remove the proximal short pin (ANC774) near to the cut. Use the countersink (ANC120-US) to prepare the first cortical.



Insert a locking screw (ST4.5LxxD) using the screwdriver (ANC975) without locking it.



Restart the step with the second short proximal pin (ANC774).




Finalize the assembly:

Insert the Ø4.5 mm compression screw (CT4.5LxxD) using the screwdriver (ANC975).

Remove the short distal pin (ANC774) et replace it by a locking screw (ST4.5LxxD).

Please, use the countersink (ANC120-US) to prepare the first cortical.

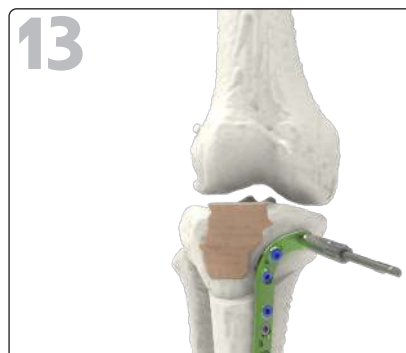

Finalize the assembly:

Drill the last two holes of the plate with the guide gauge (ANC210) and the Ø4.0 mm drill bit (ANC211).

FINAL RESULT.



Finalize the assembly:

Insert a locking screw (ST4.5LxxD) using the screwdriver (ANC975).

Please use the countersink (ANC120-US) to prepare the first cortical.

MEDIAL DISTAL FEMORAL DEROTATION - GUIDE POSITIONING (STEP 1/3)

The following surgical technique illustrates the Activmotion S medial distal femoral derotation plate (JCTxM1D). The same steps apply to the Activmotion S lateral distal femoral derotation plate (JCTxL1D).



Guide positioning:


Position the patient-matched (PM) cutting guide (ANC717) on the medial surface (lateral surface for the JCTxL1D) of the femur using the congruence of the PM cutting guide and the anterior and posterior brackets.



«Cutting K-wires» insertion:

Insert the Ø2.2 mm K-wires (33.0222.200) in the «cutting K-wire» holes.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

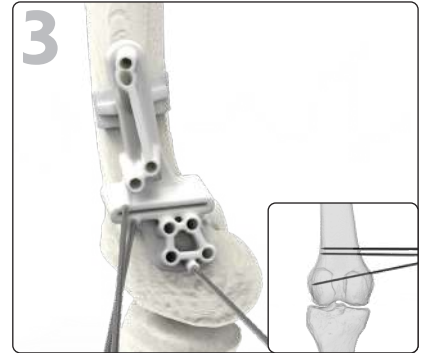


Positioning K-wire insertion:

Insert a Ø2.2 mm K-wire (33.0222.200) in the positioning K-wire hole.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.


Check the position of the PM cutting guide by comparing the fluoroscopy shot with the PM surgical technique.

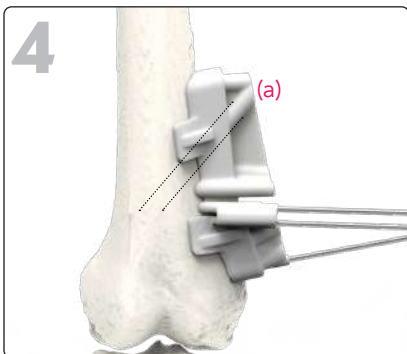

Drilling of the holes:

Drill bicortically with the Ø4.0 mm drill bit (ANC211) through the proximal holes of the PM cutting guide and monocortically (ANC211) through the distal holes.

Caution: The oriented hole^(a) on the proximal part of the PM cutting guide must be drilled monocortically.

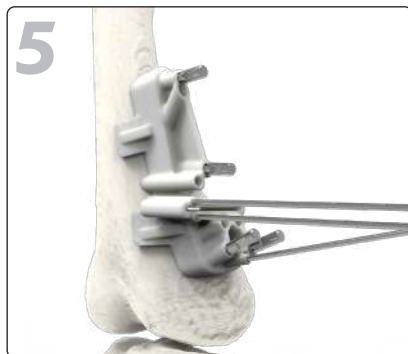

Fixation pin insertion:

Insert four long pins (ANC657), two in the distal part of the PM cutting guide and two in the proximal part, by using a mallet.

MEDIAL DISTAL FEMORAL DEROTATION - CUTTING (STEP 2/3)



Initiate the cut:

Insert the sawblade in the PM cutting guide slot and start the cut.



Remove the PM cutting guide:

Remove the long pins (ANC657), the two cutting K-wires (33.0222.200) and the positioning K-wire (33.0222.200).

Then remove the PM cutting guide (ANC717).



Finalize the cut:

Reinsert the sawblade and finalize the cut. The cut should be bicortical.

MEDIAL DISTAL FEMORAL DEROTATION - PLATE FIXATION (STEP 3/3)



9.a
Plate positioning:
 Insert two short pins (ANC774) into two proximal holes of the plate.



9.b
Plate positioning:
 The planned derotation is reached when the holes of the bone are facing the holes of the plate. Insert two short pins (ANC774) in the distal part of the plate.



10
Plate fixation:
 Press the plate against the bone and use the countersink (ANC120-US) to prepare the first cortical of the distal and proximal holes the closest to the cut.




Insert two locking screws (ST4.5LxxD) using the screwdriver (ANC975).



11
Finalize the assembly:
 Finalize the assembly by inserting the remaining Ø4.5 mm locking screws (ST4.5LxxD).

A screw must be inserted right after removing a short pin (ANC774).

Proceed by inserted alternatively proximal and distal screws getting further and further away from the osteotomy site. Use the countersink (ANC120-US) to prepare the first cortical.

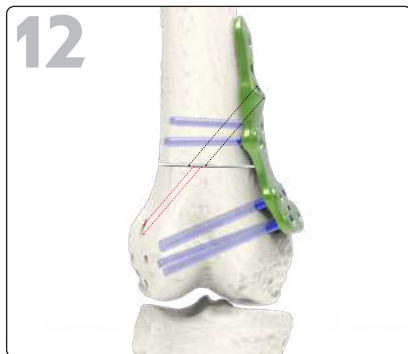

12
Insertion of the oblique screw:
 Complete the drilling of the oblique screw with the Ø3.5 mm drill bit (ANC1075) or the Ø4.0 mm drill bit (ANC1065) through the proximal hole previously performed.

Insert a non-locking screw (CT4.5LxxD) using the screwdriver (ANC975).




FINAL RESULT.



Drill the oblong hole with the guide gauge (ANC1064) and the drill bit (ANC1075). The orientation of the drill bit must be taken into account as the compression of the oblong hole is not used⁽¹⁾.

Insert a non-locking screw (CT4.5LxxD) by using the screwdriver (ANC975).

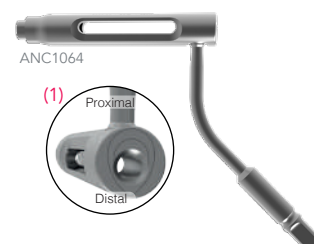

MCWDFO - GUIDE POSITIONING (PAGE 1/3)

The following surgical technique illustrate the Activmotion S medial closing wedge distal femoral osteotomy with a plate size 1 (JBTxM1D) and a femoral guide (ANC717). The same steps apply for the Activmotion S lateral closing wedge distal femoral plate (JCTxL1D) and the medial closing wedge distal femoral plate (JCTxM1D).



Guide positioning:


Position the patient-matched (PM) cutting guide (ANC717) on the medial surface of the femur (medial for the JCTxM1D and lateral for the JCTxL1D), using the congruence of the PM cutting guide and the anterior and posterior brackets.



«Cutting K-wires» insertion:

Insert the Ø2.2 mm K-wires (33.0222.200) in the «cutting K-wire» holes.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

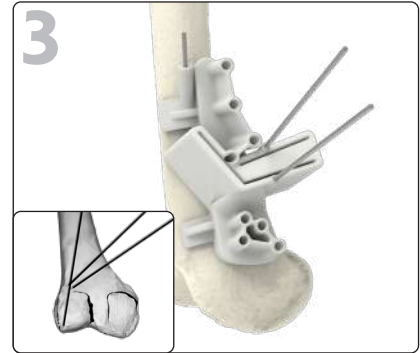

«Hinge K-wire» insertion:

Insert a Ø2.2 mm K-wire (33.0222.200) in the «hinge K-wire» hole.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

Check the position of the PM cutting guide by comparing the fluoroscopy shot with the PM surgical technique.



Drill the holes:

Drill monocortically (ANC211) through the distal holes of the PM cutting guide and bicortically (ANC211) through the proximal holes.



Fixation pins insertion:

Insert four long pins (ANC657), two in the distal part of the PM cutting guide and two in the most proximal holes, by using a mallet.

MCWDFO - CUTTING (PAGE 2/3)



Realize the first cut:

Insert the sawblade in the distal PM cutting guide slot and initiate the cut. Once the sawblade reaches the «hinge K-wire», the cut is complete.



Realize the second cut:

Insert the sawblade in the proximal PM cutting guide slot and initiate the cut. Once the sawblade reaches the «hinge K-wire», the cut is complete.



Realize the biplanar cut

Insert the sawblade in the anterior PM cutting guide slot to realize the biplanar cut.

N.B: The biplanar slot is an option and **must be selected** when the case is uploaded on the portal.



Remove the PM cutting guide:

Remove the long pins (ANC657) and the «cutting K-wires» (33.0222.200). Then remove the PM cutting guide (ANC717) by sliding it over the «hinge K-wire» (33.0222.200).



Remove the bone wedge:

Remove the bone wedge



Remove the «hinge K-wire»:

Remove the «hinge K-wire» (33.0222.200).

MCWDFO - PLATE FIXATION (PAGE 3/3)



Plate positioning:

Insert two short pins (ANC774) into the two proximal holes of the plate.



Plate positioning:

The planned closing is reached when the holes of the bone are facing the holes of the plate. Insert two short pins (ANC774) in the distal part of the plate.



Plate fixation:

Press the plate against the bone and use the countersink (ANC120-US) to prepare the first cortical of the distal and proximal holes the closest to the cut.



ANC120-US

Insert two locking screws (ST4.5LxD) using the screwdriver (ANC975) without locking them.



ANC975



Finalize the assembly:

Finalize the assembly by inserting the remaining $\varnothing 4.5$ mm locking screws (ST4.5LxD).

A screw must be inserted right after removing a short pin (ANC774).

Proceed by inserted alternatively proximal and distal screws getting further and further away from the osteotomy site. Use the countersink (ANC120-US) to prepare the first cortical.

FINAL RESULT.




LOWDFO - GUIDE POSITIONING (PAGE 1/3)

The following surgical technique illustrate the Activmotion S lateral opening wedge distal femoral osteotomy with a plate size 1(JATxL1D) and a femoral guide (ANC717).



Guide positioning:


Position the patient-matched (PM) cutting guide (ANC717) on the lateral surface of the femur using the congruence of the PM cutting guide and the anterior and posterior brackets.

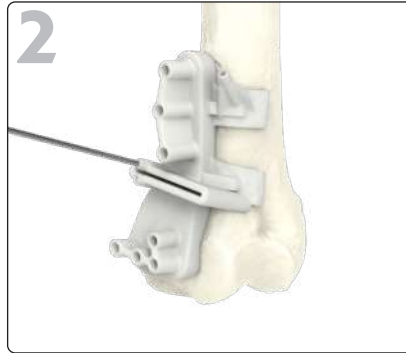

«Cutting K-wire» insertion:


Insert a Ø2.2 mm K-wire (33.0222.200) in the «cutting K-wire» hole.

To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

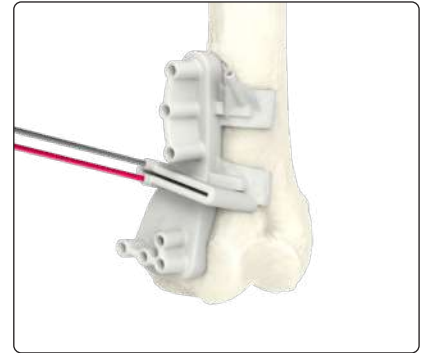

Optional - second «cutting K-wire»:

According to the PM surgical technique, a second K-wire hole can be added above the cutting slot in order to insert an optional Ø2.2 mm K-wire (33.0222.200). It will secure the sawblade path.

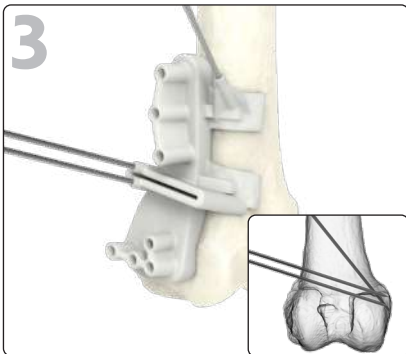


«Hinge K-wire» insertion:

Insert the Ø2.2 mm K-wire (33.0222.200) in the «hinge K-wire» hole.


To check the insertion depth, measure the part of the K-wire remaining out of the hole and compare it to the one mentioned in the PM surgical technique.

Check the position of the PM cutting guide by comparing the fluoroscopy shot with the PM surgical technique.

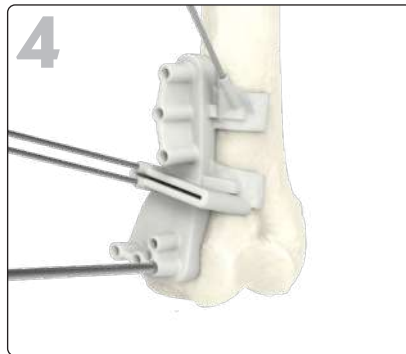

Drill the holes:


Drill monocortically (ANC211) through the distal holes of the PM cutting guide and bicortically (ANC211) in the proximal holes.

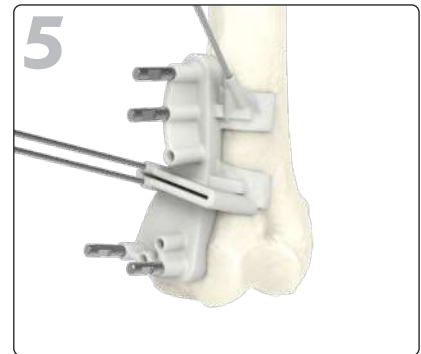

Fixation pins insertion:

Insert four pins (ANC657), two in the distal part of the PM cutting guide and two in the most proximal holes, by using a mallet.

LOWDFO - CUTTING (PAGE 2/3)

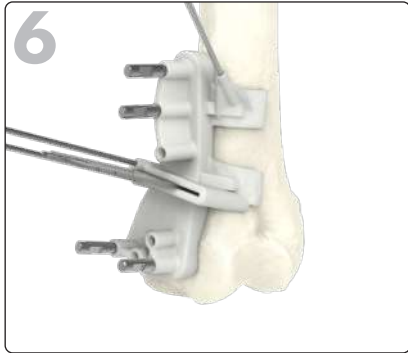

Realize the cut:

Insert the sawblade in the PM cutting guide slot and realize the cut. Once the sawblade reaches the «hinge K-wire» the cut is complete.



Remove the PM cutting guide:

Remove the long pins (ANC657) and the «cutting K-wires» (33.0222.200). Then remove the PM cutting guide (ANC717) by sliding it over the «hinge K-wire» (33.0222.200).



Perform the opening:

Insert a patient-matched wedge (ANC956) or a metallic wedge for knee osteotomy (ANC019 to ANC023 and ANC025), to perform the opening.

N.B: The patient-matched wedge is an option and **must be selected** when the case is uploaded on the portal.

LOWDFO - PLATE FIXATION (PAGE 3/3)

The following surgical technique illustrate the Activmotion S lateral opening wedge distal femoral osteotomy with a plate size 1(JATxL1D) and a femoral guide (ANC717).



Plate positioning:
Insert two short pins (ANC774) into the two proximal holes of the plate.



Plate positioning:
The planned opening is reached when the holes of the bone are facing the holes of the plate. Insert two short pins (ANC774) in the most anterior and posterior holes.



Plate fixation:
Press the plate against the bone and use the countersink (ANC120-US) to prepare the first cortical of the distal and proximal screws the closest to the cut.




ANC120-US

Insert the locking screws (ST4.5LxxD) using the screwdriver (ANC975) without locking them




ANC975

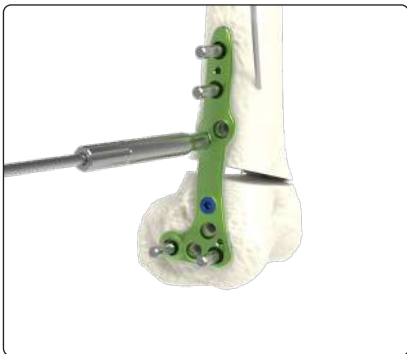

Optional - Drill the remaining holes:
According to the size of the opening or the plate position, the holes of the plate the closest to the cut might not be included on the PM cutting guide.

Drill the remaining holes using the guide gauge (ANC998) and the Ø4.0mm drill bit (ANC211). Use the countersink (ANC120-US) to prepare the first cortical.

Insert a locking screw (ST4.5LxxD) by using the screwdriver (ANC975). Remove the wedge (ANC956).

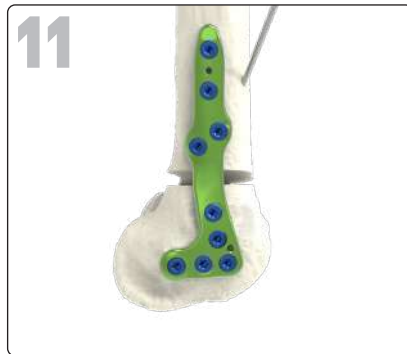

Finalize the assembly:
Finalize the assembly by inserting the remaining Ø4.5 mm locking screws (ST4.5LxxD).

A screw must be inserted right after removing a short pin (ANC774).

Proceed by inserted alternatively proximal and distal screws getting further and further away from the osteotomy site. Use the countersink (ANC120-US) to prepare the first cortical.

Then, remove the «hinge K-wire» (33.0222.200)

FINAL RESULT.



PSI references.

PSI

Ref.	Description
ANC042	Mini set - Base
ANC656	Patient specific cutting guide for HTO
ANC717	Patient specific cutting guide for DFO
ANC927	Farabeuf pliers 170 mm
ANC657	Ø3.9 mm pin for cutting guide - L60 mm
ANC774	Ø3.9 mm pin for cutting guide - L35 mm
ANC927	Farabeuf pliers 170 mm
ANC1224	Patient specific realignment guide
33.0222.200	Pin 2.2 - L200 mm

PSI OPTIONS

Ref.	Description
ANC956	Patient specific wedge
ANC1134	Patient specific cutting guide for iliac graft
ANC1135	Patient specific cutting guide for synthetic graft
ANC1184	Patient specific cutting guide for femoral head
ANC1350	Ø2.2 mm - L10 mm holding plot

OPTIONAL BONE SUBSTITUTES

Ref.	Description	Qty
0106C01	Rounded wedge 06 mm	1
0108C01	Rounded wedge 08 mm	1
0110C01	Rounded wedge 10 mm	1
0112C01	Rounded wedge 12 mm	1
1414C01	Rounded wedge 14 mm	1


Manufacturer : BIOMATLANTE (FRANCE)
Class : III
Notified body : TUV - CE 0123



Warning: All these products must be used with the Activmotion S instruments, make sure to order the NewclipTechnics kit which includes the necessary instruments.

CONTAINER REFERENCES

#	Ref.	Description
1	ANC1070/B	Activmotion S instrumentation set - Base
2	ANC1070/C	Activmotion S instrumentation set - Lid
3	ANC1070/I	Activmotion S instrumentation set - Insert

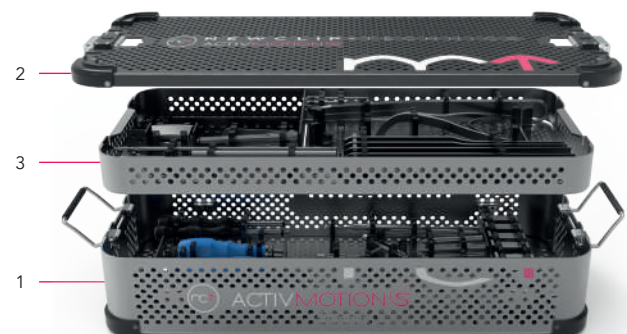


This information is intended to demonstrate the Newclip Technics portfolio of medical devices. Always refer to the package insert, product label and/or user instructions; including cleaning and sterilization before using any Newclip Technics product. These products must be handled and/or implanted by trained and qualified staff who have read the instructions before use. A surgeon must always rely on her or his own professional clinical judgement when deciding whether to use a particular product when treating a particular patient. Product availability is subject to the regulatory or medical practices that govern individual markets. Please contact your Newclip Technics representative if you have questions about the availability of Newclip Technics products in your area.

Manufacturer: Newclip Technics - Brochure EN - Activmotion S PSI - Ed.2 - 04/2026 - Medical device EC: class IIa and IIb – CE1639 SGS BE - Read labelling and instructions before the use of Newclip Technics medical devices. These products must be handled and/or implanted by trained and qualified staff who have read the instructions before use. Non-contractual pictures. Newclip Technics - 45 rue des Garottières - 44115 Haute Goulaine, France. Our subsidiaries: Newclip USA - Newclip Australia - Newclip GMBH - Newclip Japan - Newclip Iberia - Newclip Belgium - Newclip Italia - Newclip UK.

Activmotion S kit description.

#	Ref.	Description	Qty
01	ANC014-1	Osteotomy cutting guide - Part 1	1
02	ANC014-2	Osteotomy cutting guide - Part 2	1
03	ANC1069	Wedge for osteotomy - 4 mm high	1
04	ANC019	Wedge for osteotomy - 6 mm high	1
05	ANC020	Wedge for osteotomy - 8 mm high	1
06	ANC021	Wedge for osteotomy - 10 mm high	1
07	ANC022	Wedge for osteotomy - 12 mm high	1
08	ANC023	Wedge for osteotomy - 14 mm high	1
09	ANC025	Wedge for osteotomy - 16 mm high	1
10	ANC860	Wedge for osteotomy - 18 mm high	1
11	ANC024	Handle for wedges and osteotomy cutting guide	2
12	ANC120-US	Ø4.2 mm countersink with US quick coupling system	1
13	ANC210	Length gauge for Ø4.5 mm screws	1
14	ANC211	Ø4.0 mm quick coupling drill bit	3
15	ANC214	Soft tissues protector	1
16	ANC352	Ø6 mm US quick coupling handle	2
17	ANC649	Spacer Ø8 mm for HTO with ligamentoplasty	1
18	ANC601	Spacer Ø10 mm for HTO with ligamentoplasty	1

#	Ref.	Description	Qty
19	ANC621	Chisel Pauwels - 10*240 mm	1
20	ANC622	Chisel Pauwels - 25*240 mm	1
21	ANC628	Chisel Pauwels - 15*240 mm	1
22	ANC629	Chisel Pauwels - 20*240 mm	1
23	ANC975	T20 screwdriver with US quick coupling system	2
24	ANC980	T20 screwdriver with AO quick coupling system	1
25	ANC990	Osteotomy Meary pliers	1
26	ANC998	Ø4.0 mm threaded guide gauge for Ø4.5 mm screws	3
27	ANC1009	Reductor of drill guide for Ø2.2 mm pin	2
28	ANC1063	Ø4 mm mini invasive threaded guide gauge	2
29	ANC1064	Ø3.5 mm non threaded bent guide gauge	1
30	ANC1065	Ø4.0 mm quick coupling drill bit - L225 mm	2
31	ANC1075	Ø3.5 mm quick coupling drill bit - L195 mm	2
32	33.0222.200	Pin Ø2.2 L200 mm	6
/	ANC1070/B	Activmotion S instrumentation set - Base	
/	ANC1070/C	Activmotion S instrumentation set - Lid	
/	ANC1070/I	Activmotion S instrumentation set - Insert	

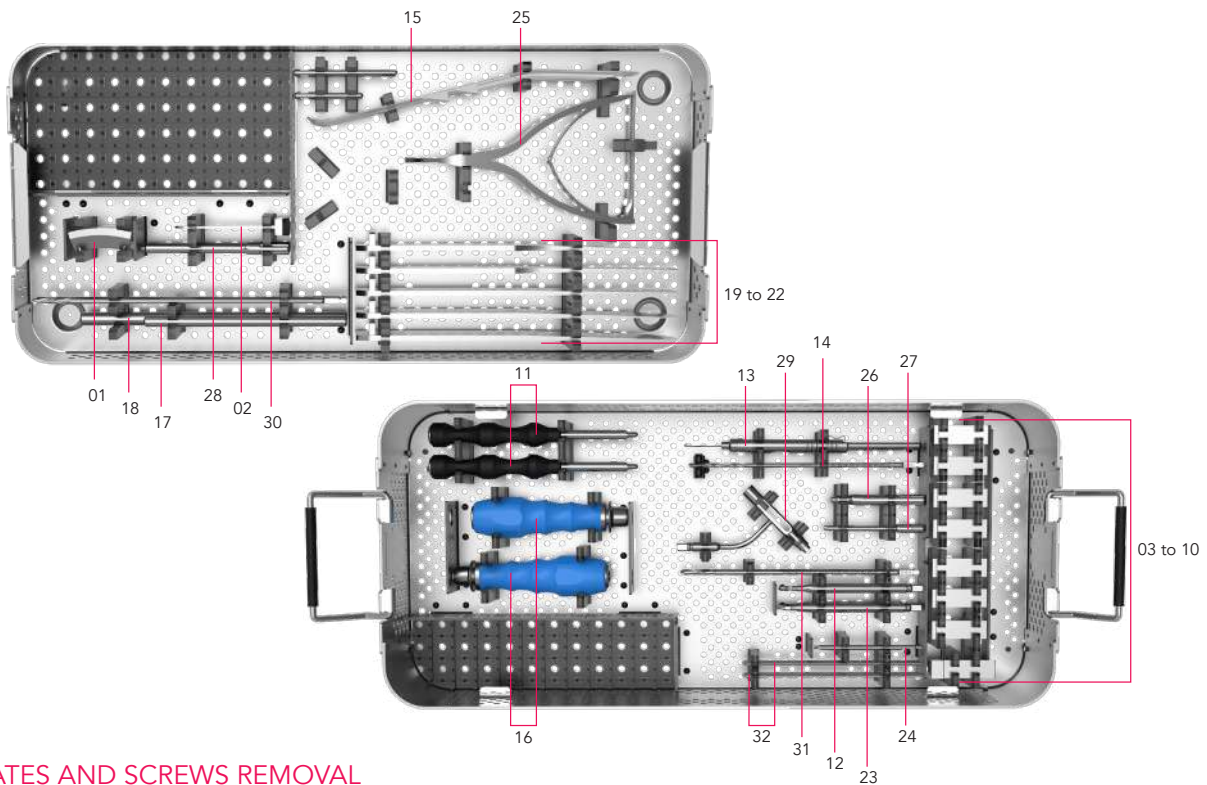

PLATES AND SCREWS REMOVAL

If you have to remove Activmotion S implants, make sure to order the Newclip Technics removal set which includes the following instruments:

- ANC042: Mini set - Base
- ANC1659: Ø6 mm US quick coupling T - handle
- ANC975: T20 screwdriver with US quick coupling system
- ANC352: Ø6 mm US quick coupling handle

An extraction set can also be ordered separately.

newcliptechnics.com

